River Valley Veterinary Clinic, S.C.

395 Main Street, Plain, WI 53577

400 S Albert Ave., Reedsburg, WI 53959

(608)546-4911 (800)546-4921 (608)524-8814 Fax (608)546-2032

rvvets@gmail.com

Michael J. Knoll, DVM
Ashley C. Kruse, DVM
Nessa S. Stika, DVM

Jill A. Janssen, DVM
Dustin A. Lochner, DVM
Erin M. Murphy, DVM

Matthew R. McGlynn, DVM
Catherine M. Bargren, DVM
Allison A. Inlow, DVM

Esophageal Feeder Protocol

1. Prior to each use, examine the feeder to make sure it's clean and there is no damage to the tube.
2. Position the calf so to prevent fluid from entering the lungs.
 - a. Tube the calf while its standing. Position its hind end in a corner and hold its head between your legs.
 - b. If the calf cannot stand, sit it up on its sternum and hold the head between your legs.
3. Compare the tube length to the distance between the calf's mouth and the point of its shoulder. The tube will be inserted that distance.
4. Clamp the flexible part of the tube closed to prevent fluid from entering the lungs while passing the tube.
5. Moisten the end of the metal bulb with colostrum to make it slippery.
6. With one hand stimulate the calf to open its mouth by placing pressure on the gums.
7. With the other hand, slowly and gently insert the metal tube until the bulb is at the back of the throat, which stimulates the calf to swallow.
8. Wait for the calf to swallow the bulb before sliding the tube down the esophagus to the measured distance.
9. Check for correct tube placement on the left side of the neck. There should be two tube-like structures: the trachea which is firm and ridged and the feeder which is hard and smooth.
 - a. If you slide the tube back and forth a few inches, you should be able to feel the bulb of the tube sliding along the neck.
 - b. If you only feel one tube, pull the tube out and try again.
10. After verifying correct tube placement, open the clamp on the tube and hold the bottle up so the fluid flows by gravity. Hold the tube and the calf's head steady while it flows.
11. When the bottle and tube are empty, clamp the tube.
12. Pull the tube out in one smooth motion.
13. Clean and disinfect the feeder and tube after each feeding. Use a clean feeder for each calf.

Dairy Herd Management • Swine Production Medicine

Ultrasound/Fetal Sexing • Small Animal Services

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